



# Model 7150-AC Wheelchair Scale Ramp and Ramp Strap Installation Instructions

## UNPACKING

Before beginning assembly, unpack all components and remove all packaging materials, protective wrapping, and filler. Verify that all items listed below are included in the carton:

Qty.	Item Description
2	7150 Ramp Weldments
2	Ramp Straps
12	.25-20 X 0.438 in Flat-Head Socket Cap Screw Zinc Plated
1	5/32 in. Short Hex Key
1	0.5 ml Packet High Strength Thread Locker

## RAMP INSTALLATION



**IMPORTANT!** Ensure the column is fully locked in the storage/transport position before proceeding.

Referring to Figure 1, perform the following steps to install the ramps.

1. Lift the scale and place the platform on its side in the **storage/transport** position. Have an assistant support the platform while the ramps are being installed.

**NOTE:** If available, place a wood block between the floor and the ramp to support the ramp during installation.

2. Attach each ramp as follows:

- Start with the top mount. Apply thread locker to a flat-head screw. Then, while holding the ramp against the top mount, hand-thread the screw into the rubber mount.
- Apply thread locker to a second flat-head screw. Then, while holding the ramp against the bottom mount, hand-thread the screw into the rubber mount.
- Using the supplied 5/32 in. Short Hex Key, tighten both flat-head screws securely.

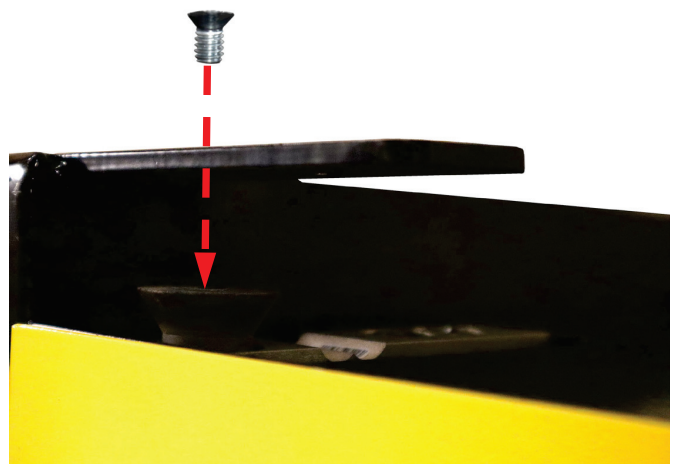


Figure 1

3. Repeat Step 2 to install the ramp on the opposite side of the platform.



**IMPORTANT!** When installing the screws into the rubber mounts, do not overtighten them, as this may damage the mounts.



# Model 7150-AC Wheelchair Scale Ramp and Ramp Strap Installation Instructions

## RAMP STRAP INSTALLATION

Referring to Figures 2 and 3 below, perform the following steps to install the ramp straps.

1. Attach the first end of a ramp strap:
  - Position the flat side of the ramp strap underneath the extended mounting tab on the ramp, aligning the strap with the two mounting holes.
  - Apply thread locker to the first flat-head screw. While holding the strap in place, hand-thread the screw through the mounting tab and into the strap. See Figure 3.
  - Apply thread locker to the second flat-head screw. Hand-thread it through the mounting tab and into the strap.
  - Using the supplied 5/32 in. Short Hex Key, tighten both screws securely.
2. Attach the opposite end of the same ramp strap to the platform using the same procedure.
3. Repeat Steps 1 and 2 for the ramp strap on the opposite side of the platform.



Figure 2

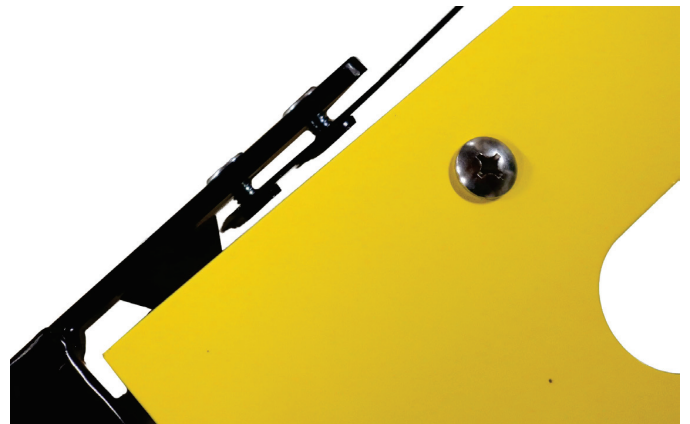


Figure 3

## DETECTO

102 E. Daugherty, Webb City, MO 64870 USA  
Ph: 417-673-4631 or 1-800-641-2008 • Fax: 417-673-2153  
[www.Detecto.com](http://www.Detecto.com)

Technical Support: Phone 1-866-254-8261 or E-mail: [tech@cardet.com](mailto:tech@cardet.com)